

Stool sampling with the SENTiFIT[®] pierceTube

It's that easy

The SENTiFIT[®] pierceTube is extremely easy and hygienic to use. The following instructions show you how to correctly take a stool sample in 7 steps.



Step 1



Place some toilet paper in the toilet bowl to collect the stool sample. You may need a larger amount of toilet paper depending on the bowl.

Note: If you received a stool collection paper please follow the instructions and jump to step 3.

Step 2



The bowel movement should be performed over the toilet paper.

Note: Your stool must NOT come into contact with urine or water in the toilet bowl. You may have to adjust your position to your toilet bowl.


Step 3



Remove the green cap from the sampling tube.

SENTiFIT® pierceTube – it's that easy

Step 4

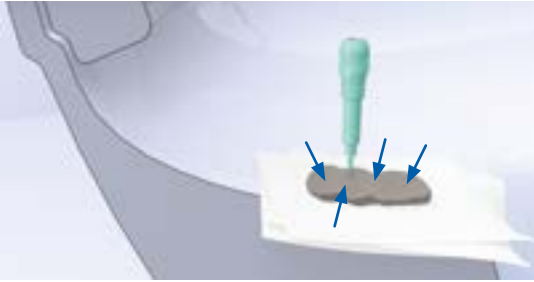

Stick the green collector stick into the sample, not too deep, at four different points.

Step 5




Scratch the surface of the stool with the green collector stick in a cross-wise motion movement.

Step 6



Ensure all the indentations are filled with stool. This is exactly the right amount.

Step 7

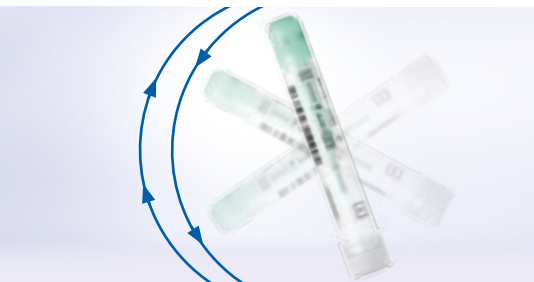

Firmly screw the green cap onto the tube again and invert gently a few times. You must NOT open it again. Return the tube no later than the next day.

Please observe the safety instructions given in the package insert.
You can view an animated version of our instructions at www.fitscreening.eu